

U-Med

Your Best Choice



Operation Manual

DROP ARM OPERATION



MAIN FUNCTION:

To adjust the armrest level, and make easy to transfer patient from chair / bed, and can prevent the helper to reduce lower back pain.

- 1) Under the armrest pad, there have push button in front of two side armrest



- 1) Push the button and hold the armrest slowly to downward the armrest
- 2) Do not put the finger or hand between the gap of armrest
- 3) Until the armrest is same level of seat pad, then can transfer the patient



- 1) After transfer the patient, we can slowly raise up the armrest
- 2) When raise the armrest at the highest point, and will hear the sound "Tic". That means the armrest will be locked in safety position



CONTROL THE BACK AND OTTOMAN W/O POSITION LOCK**MAIN FUNCTION:**

Control the seat and back angle to fit any patient requirement. Provide comfort position for different case of patient.

- 1) Before to adjust back and seat angle, we must lock the rear castor, and prevent the chair turning



- 1) To adjust the chair in TV position.
- 2) Two hands hold the push handle, or one hand hold the push handle and the other hand hold the back pad upper position, or
- 3) The user can use hand to push the armrest, and get the ottoman open and the back recline



- 1) Pull the push handle at the back, and the ottoman will be opened and the back pad will be reclined
- If the height of helper cannot easy to pull the push handle, and may be used the foot stepped on the basic back frame

CONTROL THE BACK AND OTTOMAN W/O POSITION LOCK

- 1) To adjust the chair in full recline position.
- 2) Press the control handle, and pull the back pad downward



- 1) If you feel the recline angle acceptable or comfortable, and you can release the control handle
- 2) After release the control handle, the gas lift will be locked in your comfort position





- 1) From full recline to upright position, we can press the control handle and push the back pad in normal position

CONTROL THE BACK AND OTTOMAN W/ 3 POSITION LOCK**MAIN FUNCTION:**

Provide 3 position locking system in different angle by third party, and prevent the patient to control the chair in any angle to occur any incident.

- 1) Before to adjust back and seat angle, we must lock the rear castor, and prevent the chair turning

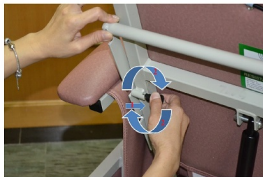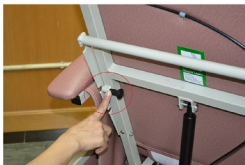

- 1) Lock in upright position.

Release the back stopper frame. Pull and rotate the stopper handle, and can get the back stopper frame release.

CONTROL THE BACK AND OTTOMAN W/ 3 POSITION LOCK**1) Lock in upright position.**

Put the stopper frame in the first lock position, and can lock in upright position

**1) Lock in TV Position**

Two hands hold the push handle, or one hand hold the push handle and the other hand hold the back pad upper position, or

The user can use hand to push the armrest, and get the ottoman open and the back recline

Put the stopper frame in the second lock position, and can lock in TV position

**Lock in Full Recline Position**

Release the back stopper frame. Pull and rotate the stopper handle, and can get the back stopper frame release.

Same procedure as TV Position, and press the control handle, and pull the back pad downward until to lock the stopper frame in the third position.



FOOTREST**1.**

Use the foot to raise little bit of the footrest and pull out.

2.

Pull out until footrest complete extend

3.

Use the foot to raise little bit of the footrest and push back to the frame

4.


Make sure the footrest is completed at the chair base

OPERATING

MEAL TRAY



Pull and rotate the hand knob in release or unlock position

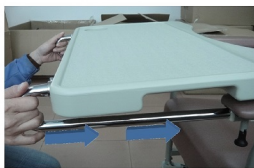

Insert the supporting arm of meal tray to the hole of armrest



Choose the distance of meal tray between the user, and rotate the hand knob in lock position



Push the supporting arm of meal tray, and will hear the "TIC" sound. That means the hand knob is locked with supporting arm.

⚠ WARNING

Carefully follow cleaning instructions provided with cleaning product.

DO NOT use paint remover or any liquid brush cleaner.

NOTE: A variety of solvents may be used to clean the upholstery on the recliner dependant upon the severity of the stain. The following list should be helpful:

- Ordinary dirt stains: Clean upholstery with warm water and mild detergent to remove superficial soil.
- Ground-in dirt stains: Use soft bristle brush and powdered cleanser or similar detergent.
- Tar, asphalt or creosote: These will stain and should be removed as soon as possible. Area can be cleaned with kerosene or naphtha.
- Chewing gum: Scrape off carefully. Remove remainder with kerosene or naphtha.
- Paint or shoe polish: Remove immediately. Use a white cloth dampened with kerosene, naphtha or turpentine.
- Excrement: Remove immediately. Clean with 1:10 mix of bleach and water.

U-Med Products Ltd

Unit 08, 12 Floor, Sun Hing Industrial Building
22 Kin Fat Street, Tuen Mun, N.T., Hong Kong SAR

Tel: 3904 0871, 3904 0872
Inquiry: info@umed.com.hk

Fax: 3747 3706, 3904 0873
Website: www.umed.com.hk