



Product Range & Specifications High Risk to BS 7177

CODE	PRODUCT	SIZE	MAX USER WEIGHT
TEBP3	Community Bed Pad 3 LAYER	100cm x 80cm (39" x 31")	108kg (17st)
TEBP4	Community Bed Pad 4 LAYER	100cm x 80cm (39" x 31")	159kg (25st)
TEBPC	Vapour Permeable Community Bed Pad Cover	100cm x 80cm (39" x 31")	n/a
TESP3	Seat Pad for Foam Cushion 3 LAYER	40cm x 50cm (16" x 20")	95kg (15st)
TESP4	Seat Pad for Foam Cushion 4 LAYER	40cm x 50cm (16" x 20")	108kg (17st)
TESPC	Vapour Permeable Seat Cover	40cm x 50cm (39" x 31")	n/a
TEM5	Full Length Mattress Overlay 4 LAYER	195cm x 90cm (77" x 35.5")	159kg (25st)
TEMC5	Vapour Permeable Full Length Cover	195cm x 90cm (77" x 35.5")	n/a
TEM7	Matt Overlay Sealed in CRIB 7 Cover	195cm x 90cm (77" x 35.5")	159kg (25st)
TEHP	Heel Pad One Size	13cm x 13cm (5" x 5")	n/a
TEPC	Pillowcase	50cm x 75cm (19.5" x 29.5")	n/a

We reserve the right to change any design or specification without prior notice



Downsview Avenue, Pulborough,
West Sussex RH20 4PS
United Kingdom
Tel:(44) (0)208 133 2851
Fax: (44) (0) 208 196 2364

email: info@danmedicasouth.co.uk
www.danmedicasouth.co.uk

**RETURNS NOT ACCEPTED AT THE
ABOVE ADDRESS... CONTACT US
FOR WAREHOUSE DETAILS**

Dealers Stamp



HIGH RISK COMMUNITY/ ACUTE PRODUCT RANGE

General User & Safety Guide



BAR CODE CODE DESCRIPTION

--	--	--





PLEASE READ THIS INSTRUCTION GUIDE CAREFULLY BEFORE USING ANY OF OUR PRODUCTS, PARTICULARLY THE **SAFETY INSTRUCTIONS** THAT MUST BE ADHERED TO, FAILURE TO DO SO COULD INVALIDATE ANY WARRANTIES

SAFETY INSTRUCTIONS

It is recommended that on receipt of any of our products it is washed to revitalise the polyester fibres that have been compressed during both the manufacturing and packaging.

Do not use the community or acute overlay for or while lifting a patient. This will damage the product and could put the patient at risk

Facial contact with a vapour permeable cover can cause suffocation and should be avoided at all times.

Always check the safe working load of the overlay is suitable for the intended user:

Community Overlay 3 layer 17st (108kg) 4 Layer 25st (159kg)

Full Length Overlay 4 Layer 25st (159kg)

DO NOT EXCEED THE MAXIMUM PATIENT WEIGHT AT ANY TIME

When using a vapour permeable cover do not allow sharp objects to penetrate the material. This will invalidate any warranties and cause problems regarding infection control.

MOVING & HANDLING: Do not drag or pull the overlay by its cover, this could cause damage and invalidate any warranties.

HEEL PADS: Risk of slipping. Do not walk while using Heel Pads, this will invalidate any warranties, cause damage and may cause harm as this product is for bed use only.

Do not Smoke or expose to Flame when using any Treat-Eezi product

We reserve the to change any design or specification without prior notice.

WARRANTY

Overlays, Covers, Pillowcases and Heel Pads are guaranteed for a period of 2 years against manufacturing defects. Please refer to Terms & Conditions on our website for further information www.danmedicasouth.co.uk

All Treat Eezi products are CE marked and comply with BS 7177 Crib 0,1&5



INSTRUCTIONS FOR USE

Community Pad Overlay & Full Length Overlay

Both these products should be placed on top of your existing mattress in the case of the Community Pad we recommend placement on the bed to cover from the shoulder blades down past the hips. Obviously the pad can be placed where it is required depending on each client's individual needs.

If a Treat Eezi full length Overlay is used be sure to secure it with the 4 retaining straps at each corner of your mattress to avoid excessive movement.

Ensure the risk factor is suitable for the intended user. To help prevent pressure damage it is important that patients reposition themselves, or are repositioned regularly. Try to avoid additional covers/padding as this may affect the Overlay's pressure reducing Qualities.

Any Overlay should be checked regularly to ensure they remain fit for purpose, clinically effective and pose no risk of infection to either patient or carer. Visual checks should be carried out regularly to identify any problems. If any signs of contamination are identified on the cover (if applicable) the Overlay should be withdrawn from use and the cover replaced.

PILLOWCASE Use cotton side facing the mattress. Lay only on double sided blue Treat Eezi material for maximum protection.

HEEL PADS Do not walk while using Heel Pads as there is a risk of slipping this will invalidate any warranties, cause damage and may cause harm as this product is for bed use only.

CLEANING & CARE

Treat Eezi Community Pads, Full Length Overlays, Heel Pads and Pillowcases can be machine washed at 80 degrees and can also be bleached. Please refer to your organisation disinfection protocol (where applicable).

DO NOT USE FABRIC CONDITIONER this can flatten the fibres and minimise effectiveness

Hang dry naturally or tumble dry on a cool setting DO NOT IRON



COVER Wipe with mild detergent wash up to 75 degrees C. The cover can be dry cleaned. To disinfect use Hypo-chlorite at no more than 1000 parts per million of active chlorine. Do not use biological or phenol-based cleaners. Wash and dry completely before storage. Some surface wrinkling may occur.

