

# Backrest

10232



## ASSEMBLY/USER INSTRUCTIONS

This product comes fully assembled.

To adjust angle of backrest, move support bar into various grooves in the frame.



Be sure support bar is securely placed into grooves, if not, the backrest can flatten/fold rapidly and cause injury to use.

## CARE INSTRUCTIONS

Clean frame with mild non-abrasive cleanser and water, then towel dry.



Maximum  
User Wt:



**Manufactured For:**

Valentine International Ltd., Taiwan R.O.C.

**Authorized European Representative:**

Medica Surgical Innovations Limited

BB2 4PB, United Kingdom

Made in Taiwan