



# Suction Grab Bar

11613/12, 11613/16, 11613/18, 11613/24

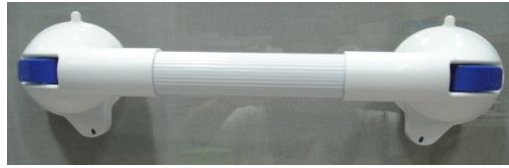


## Instructions for Use

1. Clean the tile surface where you wish to attach the grab bar.
2. Press grab bar to the cleaned area. Be sure the tile surface is thoroughly dry and rubber discs are completely touching the surface. The rubber discs should not be on or touching any grout lines or tile joints.

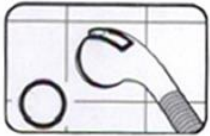

Correct installation

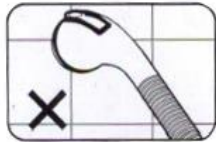

Incorrect installation  
on grout lines

3. Push the latch on each side of the grab bar into the lock position to activate suction. When suction is activated you will see a green contact indicator. If suction is not activated the indicator will be red. The grab bar is correctly activated only when contact indicator is GREEN.



Lock Position -  
Safe to Use

Green = Safe →



Release Position

Red = Not Safe →

## Removal Instructions

1. Move both latches to the release position and remove.

## Maintenance and Care

1. Check the rubber discs regularly to make sure they are clean and free of oil and dust.
2. The rubber discs should not be exposed to sunlight for long periods of time as this could cause them to lose elasticity.
3. If the rubber discs show signs of deterioration they should be replaced immediately.



The suction grab bar is designed as an aid in balance. Do not apply full user weight to the bars. Always test the grip of the grab bar on each surface and each time you use the device.

**Manufactured For:**  
Valentine International Ltd., Taiwan R.O.C.  
**Authorized European Representative:**  
Medica Surgical Innovations Limited  
BB2 4PB, United Kingdom



Made in Taiwan

Rev. 092710