



Re-Order Number	Size	Cover Fabric	Unit
554010	17"x8"x8"	LSII with Anti-Slip Mat	Pair
554020	17"x8"x8"	No Cover	Pair

### Purpose

Skil-Care's Positioning Wedge is designed to promote proper hip-spine alignment, reduce sacral pressure to prevent the formation of pressure sores, and aid in the healing of existing sores. Soft and secure, the Position Wedge is more secure than a pillow.

### Easy to Use

1. Place patient on his/side and slip tapered edge of wedge beneath patient, just below the shoulders. (Non-slip matting faces bed. It does not come in contact with patient).
2. Allow patient to roll back onto the wedge.
3. Reposition patient according to your facility's established procedures and policies.

### Easy Care

The Positioning Wedge may be wiped clean using a spray-and-wipe cleanser. Disinfect using a non-chlorine disinfectant.

### Added Feature

The Positioning Wedge now has anti-slip mats sewn to the bottom. This material keeps wedge securely in place. Make certain that the anti-slip mat is positioned so that it is in contact with the bed sheet.

