

Taylor Percussion Hammer

61001

Features a molded triangular shape rubber head and chrome plated handle with pointed tip. Also available in optional basis per different finished of handle i.e. " Duracote " Solid Color, Titanium, Satin and Matte Gold.

As this unit made from finest quality materials available and distinctive craftsmanship, with proper care and maintenance will surely provide you ageless dependable performance.


Herewith follows some useful tips how to enjoy maximum performance of the unit and ways to maintain it.

1.) Triangular Rubber Head

Urologist or practioner uses this side to check / test the patient for neurological reflexes of upper and lower extremities, which help determine and conclusion of diagnostic procedure.

2.) Metal Handle with Pointed Tip

This section is being utilized to determine sensitivity responses of patient and threshold against pain.

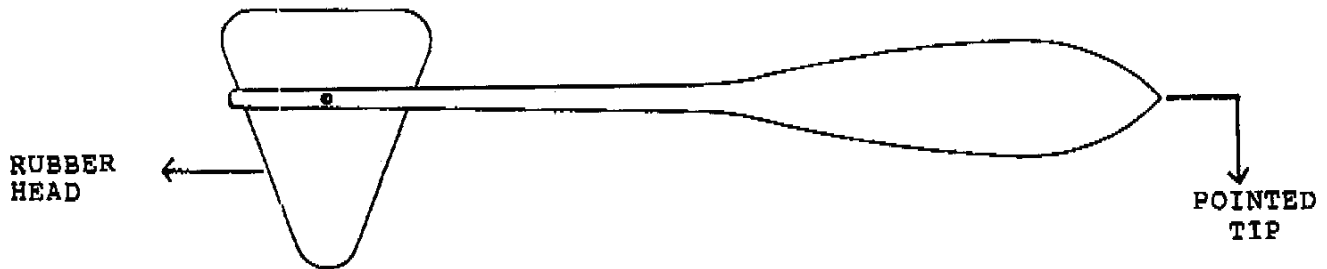

3.) Care and Maintenance

- a.) Do not expose to excessive heat or direct sunlight.
- b.) Do not remove the stopper pin holding the rubber head and handle.
- c.) Clean using alcohol by wiping with a cloth or cotton balls.
- d.) Keep away from the reach of children.



Valentine International Ltd.
8th Floor, No. 149, Sec 2, Ta Tung Road
221 Hsichih District, New Taipei City
Taiwan R.O.C

EC REP

Y. Sung Handelsvertretung
Duesselthaler Str. 24
40211 Duesseldorf Germany



Made in Taiwan

Rev. 123020