

Air Mattress Pump Instructions (APP-P06-B)

No.	Functions	Operator Instructions
1.	Auto pressure control	a. Use up/down arrow buttons to adjust air pressure to match the patient's weight. b. DO NOT touch the controls for 3 seconds to confirm settings. c. Pump will stop when the air pressure is done setting up and only restart when pressure is below normal.
2.	Alternating pressure cycle timer	a. Options available: 6, 9, 12, 25 Minutes pressure cycle time.
3.	Weight pressure control	a. Uses an electromagnetic valve control making it quieter and more durable.
4.	Auto firm (Static)	a. Inflates the mattress till it's firm, reverts to previous settings after 30 mins .
5.	Seat inflating	a. Mattress pressure increases by another 15-20% to provide better support by seating.
6.	Panel lock	a. When light is on, pump settings are locked. Hold for 3 seconds to unlock.
7.	Power switch	a. Default settings are 9 minutes cycle and 60kg pressure.
8.	LED & LCD display	a. Screen displays weight adjustment and air pressure from the pump.
9.	Remote	a. Wireless remote.
10.	Pump filter	a. Pump comes with a filter attached.
11.	Alarm	a. Alarm will go off when there is an accidental power off or disconnected plug while on. Press the alarm to turn off.
12.	Low pressure alarm	a. Alarm will go off if there is low pressure (<25mmhg) . Press the alarm to turn off.
13.	CPR latch	a. Plug in to fill with air OR release to deflate the mattress.