

# Bariatric Gel-Foam Pad



Weight-tested  
to  
500 pounds.

Reorder #	Size	Unit
<input type="checkbox"/> 915110	20" x 18" x 3 ½"	Each
<input type="checkbox"/> 915120	22" x 18" x 3 ½"	Each
<input type="checkbox"/> 915130	24" x 18" x 3 ½"	Each
<input type="checkbox"/> 915135	26" x 18" x 3 ½"	Each
<input type="checkbox"/> 915133	26" x 20" x 3 ½"	Each
<input type="checkbox"/> 915134	28" x 18" x 3 ½"	Each
<input type="checkbox"/> 915137	28" x 20" x 3 ½"	Each
<input type="checkbox"/> 915139	30" x 20" x 3 ½"	Each

## Purpose

Skil-Care's Bariatric Gel-Foam Pad features a surface layer of water-based gel that relieves interface pressure while cooling the skin to reduce perspiration. The foam base adds both comfort and stability. This cushion is specially constructed to prevent bottoming-out and maximize weight distribution.

## Important Notice

The Bariatric Gel-Foam Pad's cloth cover has safety straps that must be secured to the wheelchair to prevent the cushion (and possibly the user) from slipping off the wheelchair seat.

## Easy Care

The inner vinyl cover is heat sealed to prevent urine and other fluids from seeping into the cushion. The vinyl cover may be cleaned with a spray-type cleanser and disinfected with a non-chlorine disinfectant. The outer cloth cover may be laundered and dried at temperatures that do not exceed 180°F (82°C). Do not use bleach.

**Manufactured by: Skil-Care Corporation • Made in the U.S.A**