


GEL-GRIP

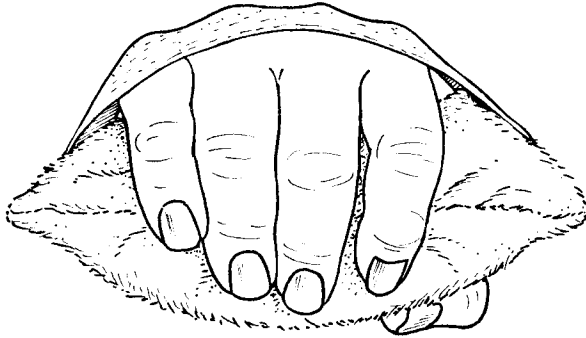

Reorder #	Size	Unit
<input type="checkbox"/> 201170	Small-Medium	6/PK
<input type="checkbox"/> 201175	Medium-Large	6/PK

Purpose


Skil-Care's Gel-Grip contains a water-based gel that provides tactile stimulation. It controls involuntary hand closure and encourages hand exercise. It may also be warmed to sooth arthritic hands.

Easy to Use

● Place patient's hand through elastic band to prevent the Gel-Grip from being dropped.

● Sooth arthritic hands by removing the gel insert (see illustration) and warming it in water that does not exceed 110°F (54°F).

Do not microwave gel insert. Place insert back in cloth cover.

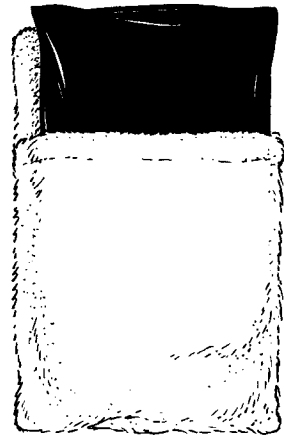

Easy Care

The cloth cover may be removed and laundered and dried at temperatures that do not exceed 180°F (82°C). The gel insert may be hand washed and disinfected with a non-chlorine disinfectant.