


**Medical chair scale**  
**ASSEMBLY & OPERATING INSTRUCTIONS**  
**RGT.B1-200-RT**

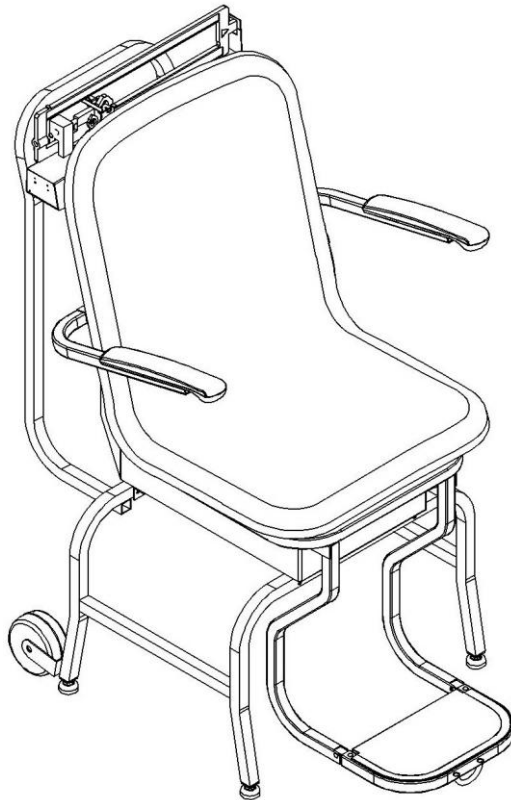

---

## ■ Specifications

Model	RGT.B1-200-RT
Max.	200kg(440lb)/160kg(350lb)/140kg(300lb)
Division	100g(1/4lb)
Class	Ⓓ
Overall dimension	870×620×1040mm (LXW XH)
Deadweight	25kg

## ■ ASSEMBLY & OPERATING INSTRUCTIONS

The precision instrument is factory pre-assembled. Before starting to use, please unpack carefully and remove all wrappings and fillers. Lay down scale. Put column horizontal to floor.

### ■ Operating Instructions

#### Zero Balance Adjustment

The scale should be checked periodically, particularly when moved, by following the instructions below:

1. Make certain scale is on a hard, level surface. Step on and off scale a few times in order to align all internal parts.
2. Place both poise weights on zero (see Fig. 1). The beam pointer should float gently up and down and not touch the top or bottom of the trig-loop.

#### Weighing Instructions

1 Set the large and small poises to indicate your approximate weight.

2 Step on the scale. Continue to adjust the small poise until the pointer is centered. NOTE: It is not necessary for you to wait for the pointer to stop moving in order to read your correct weight. As long as the pointer is moving equally in the trig-loop opening, the pointer would eventually come to rest in the center of the opening. Even when the pointer is not perfectly centered, as long as it is moving, you are within one quarter pound of your correct weight.

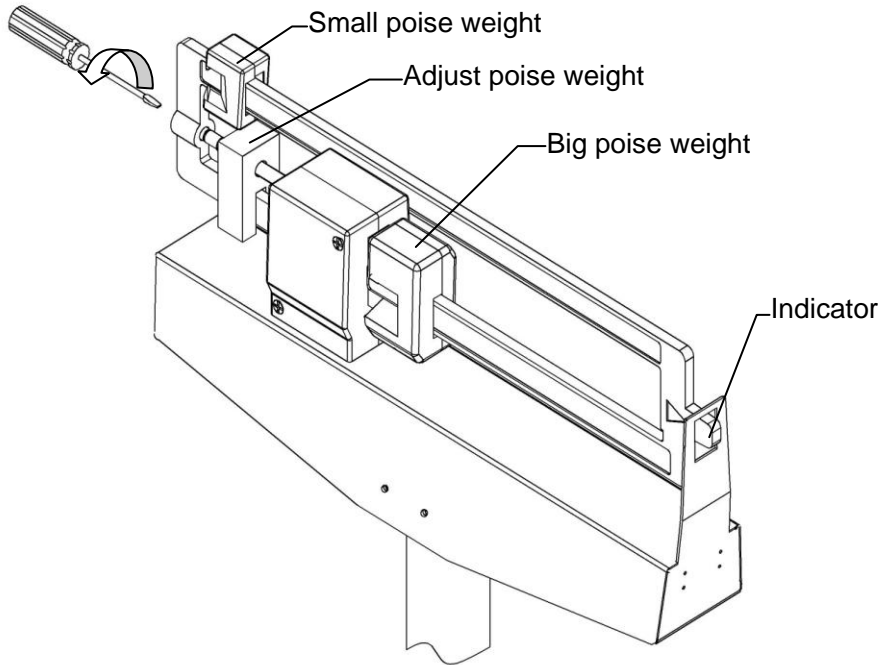


Fig.1

**3** Add the indicated reading of small poise to the indicated reading of the large poise. The total is your weight.  
locked in the original place in the range of the inner height rod and the inner height rod should be at its lowest position).

**NINGBO YIFENG LIFECARE IMPORT AND EXPORT CO., LIMITED**  
Address: 18<sup>TH</sup> FLOOR, HENGFU BUILDING 1, NO. 828 FUMING ROAD,  
NINGBO, CHINA  
TEL: 0086-574-87482339 / FAX: 8657487487171  
MOB: 13736178445 / Wechat: 13736178445  
E-mail: [sugar@yflifecare.com](mailto:sugar@yflifecare.com) / Skype: anncear