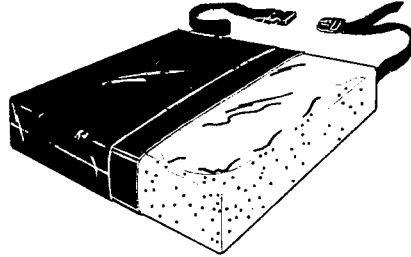


Thin-Line™ GEL-FOAM PAD

**Reorder # Size**

- | | |
|---------------------------------|-----------------|
| <input type="checkbox"/> 751039 | 16" x 16" x 1½" |
| <input type="checkbox"/> 751040 | 18" x 16" x 1½" |
| <input type="checkbox"/> 751042 | 20" x 16" x 1½" |

Skil-Care's new Thin-Line™ Cushions feature a low-profile design that enables most smaller wheelchair users to reach the floor or footrests.

Effective Design

This two-chamber wheelchair cushion combines the comfort and therapeutic effectiveness of gel and foam. The high-density foam base prevents bottoming out, provides stability, and increases comfort. The water-based gel surface cools the skin to prevent perspiration while relieving interface pressure.

New Low-Shear Cover

The outer cover is made of two-way stretch Low-Shear fabric. It moves with the wheel-chair user to reduce skin-damaging shear and friction. The Low-Shear cover is completely launderable. It may be washed and dried at temperatures that do not exceed 180°F (82°C). Do not use bleach.

Important Safety Feature

The Low-Shear cover has a safety strap that prevents the cushion from slipping off the seat. This strap must be secured to the wheelchair frame.

Easy Care

The Low-Shear cover is launderable (see above). The incontinent-proof vinyl inner cover is factory sealed to prevent urine from soaking through to the foam. Spray-and-wipe cleanser and mild disinfectant may be used to clean the vinyl.