

VERŌ

TREATMENT RECLINER

PREMIUM SERIES



Verō PRC with optional 3" dual casters, urethane arm caps and headrest cover & pillow set – shown in Bottle (85) & Mushroom (46).

Continuously evolving patient care requirements create opportunities for enhanced solutions. The Verō provides total flexibility in matching your application with exactly the features you require and the comfort your patients demand.

The Verō recliner's modular design allows clinicians to choose the best configuration to perfectly suit their specific needs:

- New Power Recline model (Verō PRC)!
- Multiple recline options and infinite position backrest promote patient independence.
- Bilateral and foot-operated controls provide clinician comfort and convenience.
- Durable construction and easy-to-clean surfaces enable increased efficiency.

**DESIGNED BY WINCO, BUILT BY YOU.
THAT'S JUST SMART.**





VERO CARE CLINER

PREMIUM SERIES

ANATOMY OF THE VERO CARE CLINER

Fastener-Free Headrest Cover

Optional for 5 Series

Push Handle

Option

Infinite Position Backrest

Power Recline

PRC Only

Gas Assist Control Lever

5X and 6X models only

Accessory Hook

Option

Power Recline Pendant

Option: PRC Only

Urethane Arm Cap

Option

Intuitive Side Tables

Option

Premium version shown

Foot Operated Swing Arm Release

Swing Arm Models only

Weight-Bearing, Retractable Foot Plate Option

5" Caster models only

Bilateral Trendelenburg Access

6 Series Models only

IV Pole + Holder

Option

Easily Accessed Heat & Massage or Heat Only Switch

Option

50+ VINYL COLOR OPTIONS AVAILABLE

Footrest with "Heel Drop" Comfort Zone

Independent Leg Rest


Durable, Easy-to-Clean, Powder-Coated Frame

180 Degree Swing Arm

Swing Arm models only

Fixed Front Casters with Accessible Lock Tabs

*5" Casters only
(Pedestal Feet or 3" Caster options available)*

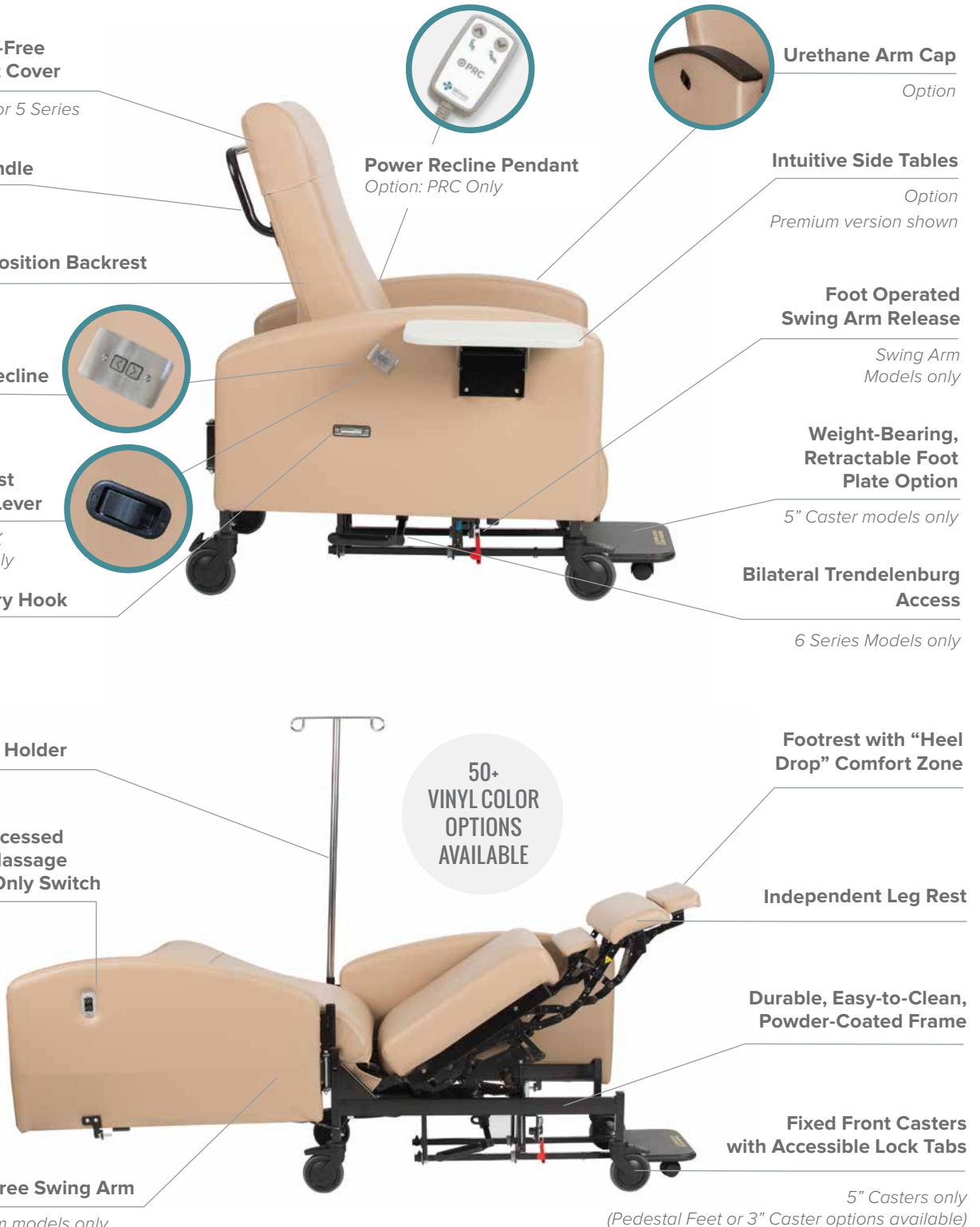

TRENDELENBURG

Verō Recliners are available with Trendelenburg (6 series) or without (5 series). On the **Verō**, Trendelenburg is controlled with bilateral access foot levers, reducing clinician's need to bend to reach controls. PRC models (with power recline) offer push-button recovery from Trendelenburg to minimize potential for clinician back strain.

Exclusive to **Verō**, Secure-Glide™ provides gentle patient positioning while avoiding injury to caregivers. The controlled descent helps eliminate any need or natural reflex to manually control the rapid patient descent in emergency situations.



Secure-Glide™ for stress-free positioning



Foot lever for exclusive Secure-Glide™ Trendelenburg descent


RECLINE CONTROL OPTIONS

All **Verō** models feature convenient and comfortable infinite positioning of the back rest. With **Verō**, you have three options available for controlling recliner positioning:

Power Easy recline and recovery from trendelenburg at the push of a button. One switch standard. Dual switches and/or pendant options available.

Gas Assist provides positive locking position of the backrest angle with gas cylinder assisted operation. Locking action is controlled by easily accessed bilateral control levers.

Push-Back allows seamless positioning of the desired back rest angle. The gravity operated, gentle resistance designed into the chair maintains selected positioning.

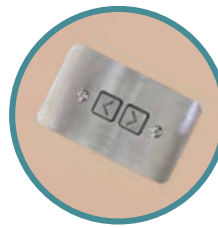

Power Recline
PRC models



Gas Assist
5X and 6X models



Push Back
5X and 6X models

ARM OPTIONS

Functionality is enhanced through the availability of two arm configurations. Standard fixed arms are cost-effective, sturdy and durable. Optional swing arms allow 180 degree movement of the arm for convenient cleaning access and barrier-free patient transfer.

Swing Arm Option Swing arms are ideal when patient transfer and easy access are important considerations. Each side of the **Verō** is easily opened with convenient foot controls.

Fixed Arm Option Fixed arms are suitable for most general purpose operations when transfer of patient is not required or where working space is limited.



Upholstered Swing Arm Shown

ARM CAP OPTIONS

Urethane Cap Option Urethane arm caps provides a unique, upscale appearance and added durability.

Urethane Arm Cap
Option



MATCH YOUR PURPOSE

The **Verō** modular design allows choices in how the chair is anchored or moved. Whether stationary or for patient transport*, **Verō** has a perfect solution.

Pedestal Feet When mobility is not required or desired, pedestal feet provide a less clinical look and a lower seat height.

Casters provide ease of movement for cleaning, maintenance and staging. Two size options are available. Both include lockable, foot lever release controls.

3" Casters Lower seated height with the flexibility of mobility. Dual wheel casters available for PRC (power) models only.

5" Casters are the optimal choice for ease in movement.



It is recommended that after moving the chair always lock the casters prior to patient ingress/egress.

**PRC (power) models only*

FOOT PLATE

Patients will appreciate the added comfort of the retractable foot plate.

The optional footplate is available for models with 5" casters. Its sturdy design is weight-bearing.

The footplate locks as weight is applied to prevent unwanted or unexpected movement.

Note: Patients should be assisted at all times by a staff member when using the footplate.



Footplate is easily positioned by foot.

PUSH HANDLE OPTION

The push handle option is recommended in combination with footplate models if the chair is to be used for in-facility patient transport*.

**Only 5X, 6X or PRC models with footplate and push handle are suitable for patient transport. See User Manual for important details.*



SIZE / CAPACITY OPTIONS FOR MAXIMIZED FLEXIBILITY

The **Verō** is available in two sizes: standard and X-large. No matter which size is specified, **Verō** offers comfort, beauty and durability.

- Standard seat width is 22.5" between arms with a 350 pound weight capacity. This chair makes petite to average size patients feel right at home.
- X-large seat width is 26.5" between arms with a 500 pound weight capacity, making it suitable to care for Bariatric patients and provide a comfortable experience.



X-Large 500 lb Capacity



Standard 350 lb Capacity

TABLE OPTIONS

There are three table options; standard, pivot and premium. All tables on the **Verō** are available in left, right or both configurations and can be ordered in a variety of combinations to suit facility requirements and clinician preference. All feature a stain resistant, non-porous material and all but the pivot table have a lip edge design to contain small spills.



Standard Table



Pivot Table



Premium Table

Each standard table adds 2.75" (2.5" for XL) to the overall chair width when lowered. Each premium table adds 1.25" (1" for XL) to the overall chair width when lowered. This added width should be considered when ordering table options appropriate for your facility.

- **Standard table** This value priced lift-to-raise-and-lower table will be familiar to current Winco Elite series owners.
- **Pivot table** swivels so patient can have the table in front of them.
- **Premium table** features an intuitive single hand up/down feature with audible positive lock and release.

COMPLEMENT DESIGN & UTILITY


- **Heat Only Option** provides three patient-selected soothing warmth ranges for maximum comfort.
- **Heat and Massage Option** adds a selection of five gentle vibrating massage modes to the heat option, enhancing patient comfort level.
- **Optional Accessory Hook** The stainless steel hook remains flush until needed, making it a convenient choice for a Foley bag. Available for either left, right or both sides.
- **Optional IV Pole and Holder** for either left, right or both sides.
- **Headrest Cover and Armrest Covers** Covers extend the life of the headrest and armrests and are fastener-free, making them easy to lift on and off.



Heat & Massage
3 levels of heat, 5 levels of massage

Heat Only
Low/Med/High

CONVENIENTLY
PLACED CONTROLS
FOR PATIENT
INDEPENDENCE

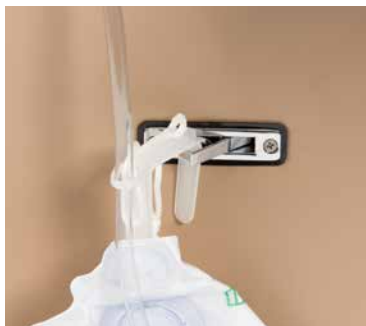

Accessory Hook
Shown here with Foley bag



IV Holder Mount
Available for left, right or both sides

MODELS & SPECIFICATIONS

| MOBILITY OPTION | PEDESTAL FEET | | | | 3" CASTERS | | | | 5" CASTERS | | | |
|---|---|--|---|--|---|--|---|--|---|--|---|--|
| | STANDARD | | X-LARGE | | STANDARD | | X-LARGE | | STANDARD | | X-LARGE | |
| RECLINER SIZE | STANDARD | | X-LARGE | | STANDARD | | X-LARGE | | STANDARD | | X-LARGE | |
| MODEL OPTION | PUSH BACK/ GAS BACK | POWER | PUSH BACK/ GAS BACK | POWER | PUSH BACK/ GAS BACK | POWER | PUSH BACK/ GAS BACK | POWER | PUSH BACK/ GAS BACK | POWER | PUSH BACK/ GAS BACK | POWER |
| SEAT HEIGHT TO FLOOR | 19.25 in (48.9 cm) | 20.25 in (51.4 cm) | 19.25 in (48.9 cm) | 20.25 in (51.4 cm) | 20.5 in (52.1 cm) | 20.75 in (52.7 cm) | 20.5 in (52.1 cm) | 20.75 in (52.7 cm) | 22.75 in (57.8 cm) | 23.75 in (60.3 cm) | 22.75 in (57.8 cm) | 23.75 in (60.3 cm) |
| SEAT WIDTH | 22.5 in (57.2 cm) | 22.5 in (57.2 cm) | 26.5 in (67.3 cm) | 26.5 in (67.3 cm) | 22.5 in (57.2 cm) | 22.5 in (57.2 cm) | 26.5 in (67.3 cm) | 26.5 in (67.3 cm) | 22.5 in (57.2 cm) | 22.5 in (57.2 cm) | 26.5 in (67.3 cm) | 26.5 in (67.3 cm) |
| SEAT DEPTH | 22 in (55.9 cm) | 23.5 in (59.7 cm) | 22 in (55.9 cm) | 23.5 in (59.7 cm) | 22 in (55.9 cm) | 23.5 in (59.7 cm) | 22 in (55.9 cm) | 23.5 in (59.7 cm) | 22 in (55.9 cm) | 23.5 in (59.7 cm) | 22 in (55.9 cm) | 23.5 in (59.7 cm) |
| OVERALL HEIGHT | 44.75 in (113.7 cm) | 44 in (111.8 cm) | 44.75 in (113.7 cm) | 44 in (111.8 cm) | 46 in (116.8 cm) | 44.5 in (113 cm) | 46 in (116.8 cm) | 44.5 in (113 cm) | 48.25 in (122.6 cm) | 47.5 in (120.7 cm) | 48.25 in (122.6 cm) | 47.5 in (120.7 cm) |
| OVERALL WIDTH (FIXED ARMS / NO TABLES)* | 29.5 in (11.61 cm) | 29.5 in (11.61 cm) | 34 in (13.39 cm) | 34 in (13.39 cm) | 29.5 in (11.61 cm) | 29.5 in (11.61 cm) | 34 in (13.39 cm) | 34 in (13.39 cm) | 29.5 in (11.61 cm) | 29.5 in (11.61 cm) | 34 in (13.39 cm) | 34 in (13.39 cm) |
| SEAT BACK HEIGHT | 29 in (73.7 cm) | 28.75 in (73 cm) | 29 in (73.7 cm) | 28.75 in (73 cm) | 29 in (73.7 cm) | 28.75 in (73 cm) | 29 in (73.7 cm) | 28.75 in (73 cm) | 29 in (73.7 cm) | 28.75 in (73 cm) | 29 in (73.7 cm) | 28.75 in (73 cm) |
| OVERALL LENGTH | Upright: 39.5 in (100.3 cm) Reclined: 69 in (175.3 cm) | Upright: 41 in (104.1 cm) Reclined: 69.25 in (175.9 cm) | Upright: 39.5 in (100.3 cm) Reclined: 69 in (175.3 cm) | Upright: 41 in (104.1 cm) Reclined: 69.25 in (175.9 cm) | Upright: 39.5 in (100.3 cm) Reclined: 69 in (175.3 cm) | Upright: 41 in (104.1 cm) Reclined: 69.25 in (175.9 cm) | Upright: 39.5 in (100.3 cm) Reclined: 69 in (175.3 cm) | Upright: 41 in (104.1 cm) Reclined: 69.25 in (175.9 cm) | Upright: 39.5 in (100.3 cm) Reclined: 69 in (175.3 cm) | Upright: 41 in (104.1 cm) Reclined: 69.25 in (175.9 cm) | Upright: 39.5 in (100.3 cm) Reclined: 69 in (175.3 cm) | Upright: 41 in (104.1 cm) Reclined: 69.25 in (175.9 cm) |
| ARM HEIGHT | 25.5 in (64.8 cm) | 25.5 in (64.8 cm) | 25.5 in (64.8 cm) | 25.5 in (64.8 cm) | 26.75 in (67.9 cm) | 26 in (66 cm) | 26.75 in (67.9 cm) | 26 in (66 cm) | 29 in (73.7 cm) | 29 in (73.7 cm) | 29 in (73.7 cm) | 29 in (73.7 cm) |
| WEIGHT (NO OPTIONS)* | Push Back: 104 lb (47.17 kg) Gas Back: 108 lb (48.99 kg) | 140 lb (63.5 kg) | Push Back: 122 lb (55.31 kg) Gas Back: 126 lb (57.15 kg) | 158 lb (71.67 kg) | Push Back: 104 lb (47.17 kg) Gas Back: 108 lb (48.99 kg) | 140 lb (63.5 kg) | Push Back: 122 lb (55.31 kg) Gas Back: 126 lb (57.15 kg) | 158 lb (71.67 kg) | Push Back: 104 lb (47.17 kg) Gas Back: 108 lb (48.99 kg) | 140 lb (63.5 kg) | Push Back: 122 lb (55.31 kg) Gas Back: 126 lb (57.15 kg) | 158 lb (71.67 kg) |
| WEIGHT CAPACITY | 350 lb (158.8 kg) | 350 lb (158.8 kg) | 500 lb (226.8 kg) | 500 lb (226.8 kg) | 350 lb (158.8 kg) | 350 lb (158.8 kg) | 500 lb (226.8 kg) | 500 lb (226.8 kg) | 350 lb (158.8 kg) | 350 lb (158.8 kg) | 500 lb (226.8 kg) | 500 lb (226.8 kg) |

OPTIONS

| | BATTERY BACKUP | DUAL SWITCHES | WIRED PENDANT | SWING ARMS | STANDARD TABLE L, R or Both Sides | PREMIUM TABLE L, R or Both Sides | FOOTPLATE | HEAT | HEAT & MASSAGE | IV POLE L, R or Both Sides | PUSH HANDLE |
|---------------------------|-------------------|-------------------|-------------------|--|---|---|-------------------------|------------|----------------|-------------------------------|--|
| AVAILABLE FOR | POWER MODELS ONLY | POWER MODELS ONLY | POWER MODELS ONLY | ALL MODELS | ALL MODELS | ALL MODELS | 5 in CASTER MODELS ONLY | ALL MODELS | ALL MODELS | ALL MODELS | <i>Recommended for Footplate models only</i> |
| *ADDITIONAL WEIGHT | -4.75 lb | - | - | - | +5 lb | +5 lb | -10 lb | -4 lb | -4 lb | +5 lb | -2 lb |
| *ADDITIONAL OVERALL WIDTH | - | - | - | Open 180°: -28.5 in (standard) / -32.5 in (XL) | Table down: -2.75 in (standard) / -2.5 in (XL) Table up: -13 in (standard) / -12.75 in (XL) <i>Measurements are per table</i> | Table down: -1.25 in (standard) / -1 in (XL) Table up: -11 in (standard) / -10.75 in (XL) <i>Measurements are per table</i> | - | - | - | - | - |

Additional options are available. Please contact us to ask about our full line of options.

WARRANTY Contact Winco or visit our website for warranty details.

COLORS Contact Winco or visit our website for available color selections.

NOTE Winco reserves the right to change specifications without notice.

CERTIFICATIONS

- 1) Winco recliners meet ANSI/BIFMA X5.4 for reliability testing
- 2) Winco foam meets California Technical Bulletin 117, Flammability Test Procedure for Resilient Filling Materials used in Upholstered Furniture
- 3) NR or TB options are available. NR meets AB2998 regulations and TB meets NFPA 101 and ASTM E1537-16 Standards for Upholstered Furniture
- 4) PRC models and Heat and Massage, Heat Only Options: Meet electrical safety standards, UL 1647 and CAN/ CSA-C22.2 No.68-92.
- 5) All individual electrical components for Heat and Massage / Heat Only options and power recline certified to IEC 60601-1.



CONTACT US

To arrange a demonstration or to speak with one of our associates, please feel free to contact us at:
800-237-3377 | WincoMfg.com | 5516 SW 1st Lane, Ocala, FL 34474 | Info@WincoMfg.com