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## 1. Specifications and Scope of Supply

**Item no.:** PMF-9185  
**ILN:** 4250355391858  
**Dimensions:** 180 x 65 cm  
**foldable :** 95 / 95 cm

**Material Rollboard:** Polyethylene, latex free  
**Material cover:** Nylon (coated), latex free

**Colour Rollboard:** Graphite  
**Colour cover:** red

**Patent no.:** P102009048319-5

**X-ray compatible, German TÜV approved**

**Scope of supply:**

- Rollboard incl. cover
- Manual

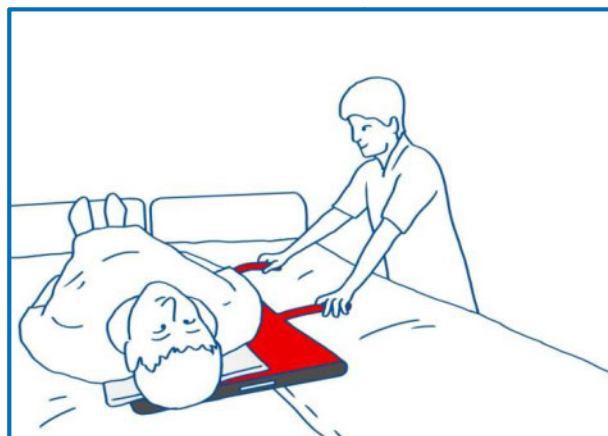
**Made in Germany**

Cover:

 max.  
200 kg

## 2. Use / Placement

**This product is suitable for re-use.**

The Rollboard Vision is ideally suited for atraumatic, painless and low-friction transfer of seated and recumbent, e. g. heavy, particularly helpless or premedicated patients to a mobile shower trolley, a different bed, a trolley, different types of wheelchairs, an x-ray, examination or operating table.

Transfer is also possible over a short distance or over different heights. The Rollboard is extremely flexible with its support for the transfer of patients with many drainages or with problems such as respiratory distress syndrome. For safety reasons, transfers with the Rollboard should be done with two persons and if applicable with a sheet for pulling the patient.

Move the patient in a recumbent position slightly to the side and slide the Rollboard about 1/5 of the way under the patient. Place the patient in a supine position on the Rollboard. Pull the patient over and onto the Rollboard in the transfer direction with the pull sheet. Without the pull sheet stand beside the transfer level and push the patient onto the Rollboard in the transfer direction.

**WARNING:**

**Ensure the working heights of the transfer levels are correct for working without back injury.**

To remove the Rollboard place the patient briefly on the side and remove the Rollboard from under the patient. The Rollboard can be used by four carers when transferring very heavy persons. The Rollboard can also be briefly lifted with a load during a transfer to negotiate an obstacle. The Rollboard Vision can be pulled up or down, moved or briefly lifted to move it away from an unfavourable transfer situation. Always be aware of your back when working as a carer.

### 3. Cleaning, Care and Disinfection

---

All Alpha<sup>®</sup> Repositioning and Transfer aids can be cleaned with a damp cloth. Use warm water and mild detergent. Wipe the product with a damp cloth if necessary and then allow to dry it thoroughly. Do not use any abrasive or bleaching agents and sharp objects.

If required, we recommend a wipe disinfection, e.g. with the product Trionic<sup>®</sup> (German DGHM/VAH approval, further information on request).

Please pay attention to the care instructions on the label of each product as well as the information on the respective instructions.

#### 3.1 Cover

---

The cover can be washed in a washing machine up to 60° C and is suitable for low dryer temperatures (45 - 50°C). Use mild detergent!

When using non-standard washing agents, bleaching agents, softener or other chemical washing additives, the functional properties of the cover can strongly degrade.

Basically, textile materials and similar surfaces are not suitable for the use of alcoholic and / or aldehyde-containing disinfectants. Do not autoclave the product!

#### 3.2 Board

---

The board itself is very durable and can be cleaned with standard household cleansing products (see item 3.). The Board may easily be disinfected with alcohol-based disinfectants as well as with Trionic<sup>®</sup> (German DGHM/VAH approval, further information on request). Allow the board to dry after disinfection.

It is not recommended to use the autoclave due to the thermo-sensitive structure of the material.

### 4. Warranty

---

- Petermann gives a warranty that the product is devoid of material defects and manufacturing defects.
- The warranty is valid for a period of **3 years**. It does not cover any defects which are due to improper use.
- Petermann cannot be held liable for any accidental or indirect injury or damage.

### 5. Visual and Function Inspection / Safety Information

---

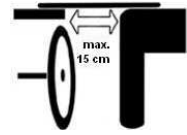
Repositioning products are also subject to natural aging. Check the equipment regularly and carefully to make sure it is working properly. Make sure that the product is in good condition and that the material, handles for pulling and seams are not damaged. Check the function of the product by applying pressure and loading to the handles and make sure that the material resists the load and does not break or tear.

Check the stability by carefully bending the board and make sure that it is elastic and also solid and stable. The smooth surface of the low-friction material makes it easy to slide and move the board. The best way to test these properties is by comparing the product with a new product.

#### **WARNING:**

- **Faulty products must no longer be used.**
- **Make sure that there are a sufficient number of carers and assistants corresponding to the transfer situation.**
- **Failure to follow these instructions may cause damage to the product or may cause injuries to you or to others.**
- **The Rollboard should only be used by trained staff.**
- **If the patient's mobility is restricted, it is essential to supervise the motions of transfer by providing trusted help.**
- **The smooth surface considerably minimises resistance to friction during the change of position. Carefully check the two transfer surfaces, making sure they are stable and brakes are applied (wheelchair, bed).**
- **Never leave the patient unattended.**
- **Particularly when lifting the Rollboard make sure that the patient is securely on the Rollboard and cannot slide off.**
- **Make sure that carers are always aware of their backs during the transfer.**
- **Do not use sharp-edged items when using the Rollboard. The Rollboard could get damaged in this case.**

- *Transfers under uncovered skin must be done with a cloth between the Rollboard and the patient (e. g. towel).*
- *When using non-standard washing agents, bleaching agents, softener or other chemical washing additives, the functional properties of the cover can strongly degrade.*
- *Make sure the Rollboard is stable on both sides of the transfer surfaces. The distance between free transfer surfaces using the Rollboard must not be more than 15 cm.*

## 6. Product Labeling

---

The Petermann Alpha® transfer aid Rollboard Vision is a medical product, class I and complies with the MDD directive 93/42/EEC.

Petermann GmbH is certified in accordance with the directive DIN EN ISO 13485 by the German TUV.

## 7. Disposal Information

---

For the Rollboard Vision there are no special disposal rules; it can be disposed of in accordance with national and local regulations as necessary.

If you have any questions about the use of our products, please call our service number:

**+49 (0) 9868 9339-0**

Petermann GmbH • Ziegelhaus 12 / postal address: Waldweg 3• D-91601 Dombühl

Phone: +49 (0) 9868 9339-0 • Fax: +49 (0) 9868 9339-20

E-mail: petermann@pm-med.de • www.pm-med.de

"Information subject to technical modifications to our products and deviations in colour or printing errors"

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Petermann GmbH est certifié par le TUV conforme à la directive DIN EN ISO 13485.
Petermann GmbH • Ziegelhaus 12 / adresse postale: Waldweg 3 • D-91601 Dombuehl