

EasyBelt/FlexiBelt

SystemRoMedic™



User manual – English

Bruksanvisning – Svenska

Brukermanual – Norsk

Brugsvejledning – Dansk

Käyttöohje – Suomi



Gebrauchsanweisung – Deutsch

Handleiding – Nederlands

Manuel d'utilisation – Français

Manuale utente – Italiano

Manual de usuario – Español

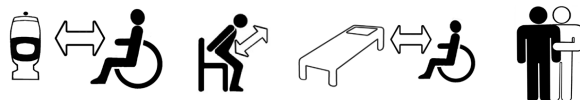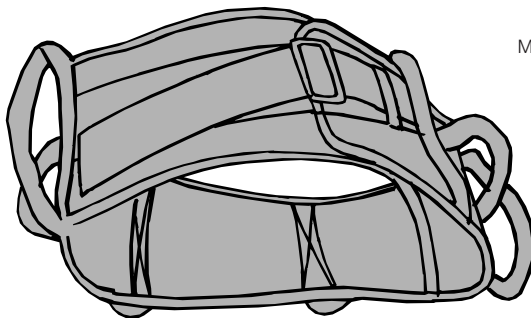

SWL: 170 kg/375 lbs

Table of contents

English	3-4
Svenska	5-6
Norsk	7-8
Danske	9-10
Soumi	11-12
Deutsch	13-14
Netherlands	15-16
Français	17-18
Italiano	19-20
Español	21-22
<hr/>	
Product information	23
Accessories	23

EasyBelt and FlexiBelt are used as support in different transfer situations. They are not to be used as assistive devices for lifting. The inside of EasyBelt is lined with nylon, making it easy to launder or wipe clean with disinfectant. FlexiBelt's soft inside feels comfortable against the skin. Both materials have an anti-slip function.




Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

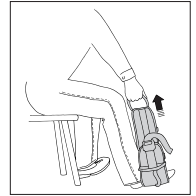

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.


Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.handicare.com.

Placement


Apply the belt around the user, below the waist.

Fasten the belt and tighten it.

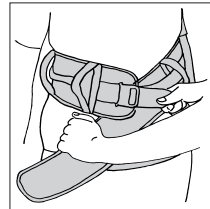


Removing

Release the clasp, first loosening any straps, and then remove the belt.

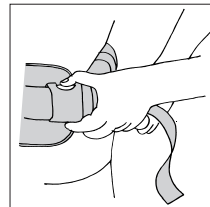


Using

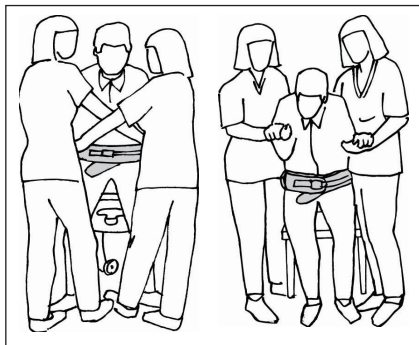


When standing up from sitting to standing the caregiver / caregivers can choose to work from the side or from the front of the user. One advantage of working in front of the user is to give a feeling of a more secure and safe standing up. In raising with two helpers, it is advisable to work with cross-grip front of the user.

It is important to lean the user forward to get a natural pattern of movement, working in towards the body with the belt.

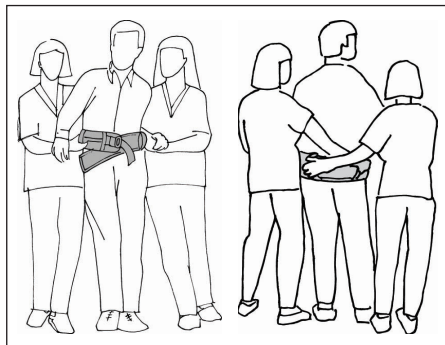

When two caregivers are assisting, it is appropriate if one caregiver stands on each side, grasping the cross-grips at the back and holding the user by the hand at the front. Cross-grips at the back provide support and enable a good working stance.

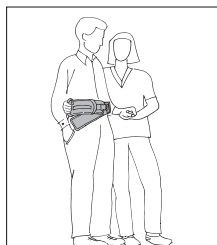

Caregivers can work in pairs or alone, depending on the user's needs, since EasyBelt/FlexiBelt have several vertical and horizontal handles.

Material

EasyBelt: Polyamid, nylon

FlexiBelt: Polyamid, polyester

Care of the product

Read the product label.



Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

Expected lifetime

Up to 10 years of normal use

Product information

Art. no.	Model	Size mm/inch	Waist measurement mm/ inch	Handles	Material, inside
6012	EasyBelt XS	750 x 120/ 29.5 x 4.7	450-700/ 17.7 - 27.5	5	Nylon
6013	EasyBelt S	1050 x 120/ 41.3 x 4.7	600-1000/ 23.6 - 39.3	7	Nylon
6014	EasyBelt M	1200 x 120/ 47.2 x 4.7	700-1200/ 27.5 - 47.2	9	Nylon
6015	EasyBelt L	1350 x 120/ 53.1 x 4.7	1000-1600/ 39.3 - 62.9	11	Nylon
6002	EasyBelt, hug XS	750 x 120/ 29.5 x 4.7	450-700/ 17.7 - 27.5	3 + 2 movable	Nylon
6003	EasyBelt, hug S	1050 x 120/ 41.3 x 4.7	600-1000/ 23.6 - 39.3	4 + 3 movable	Nylon
6004	EasyBelt, hug M	1200 x 120/ 47.2 x 4.7	700-1200/ 27.5 - 47.2	5 + 4 movable	Nylon
6005	EasyBelt, hug L	1350 x 120/ 53.1 x 4.7	1000-1600/ 39.3 - 62.9	6 + 5 movable	Nylon
6006	EasyBelt, hug XL	1500 x 120/ 59 x 4.7	1150-1750/ 45.2 - 68.8	7 + 6 movable	Nylon
6023	FlexiBelt S	1050 x 120/ 41.3 x 4.7	600-1000/ 23.6 - 39.3	7	Polyester
6024	FlexiBelt M	1200 x 120/ 47.2 x 4.7	700-1200/ 27.5 - 47.2	9	Polyester
6025	FlexiBelt L	1350 x 120/ 53.1 x 4.7	1000-1600/ 39.3 - 62.9	11	Polyester
6026	FlexiBelt XL	1500 x 120/ 59 x 4.7	1150-1750/ 45.2 - 68.8	13	Polyester
6072	FlexiBelt, hug XS	750 x 120/ 29.5 x 4.7	450-700/ 17.7 - 27.5	3 + 2 movable	Polyester
6073	FlexiBelt, hug S	1050 x 120/ 41.3 x 4.7	600-1000/ 23.6 - 39.3	4 + 3 movable	Polyester
6074	FlexiBelt, hug M	1200 x 120/ 47.2 x 4.7	700-1200/ 27.5 - 47.2	5 + 4 movable	Polyester
6075	FlexiBelt, hug L	1350 x 120/ 53.1 x 4.7	1000-1600/ 39.3 - 62.9	6 + 5 movable	Polyester
6076	FlexiBelt, hug XL	1500 x 120/ 59 x 4.7	1150-1750/ 45.2 - 68.8	7 + 6 movable	Polyester

Accessories

SupportStraps

SupportStraps can be used as a leg harness in combination with EasyBelt/FlexiBelt.

