

TOPMOST® Topmost Medical Technology Co.,Ltd

Address: No.94, Middle of GaoMing Avenue, YangHe , GaoMing , Foshan city, China.

Owner's Handbook


Please read this manual before operation for maintenance instructions and safe usage of this bed

**Contents**

1. Introduction
2. Parts Description
3. Operation of the Head and Foot Boards
4. Operation of the Bed Rails
5. Using the Motor Control
6. Care and Maintenance
7. Specification
8. Warranty

INTRODUCTION

This Electric Bed is designed for use indoors. The bed is designed for a single user of up to 250kg. The bed is available with 5" luxury casters central brake system. The bed should be moved in the flat floor and can be moved to all directions. The bed is electrically used by the patient/nurse.

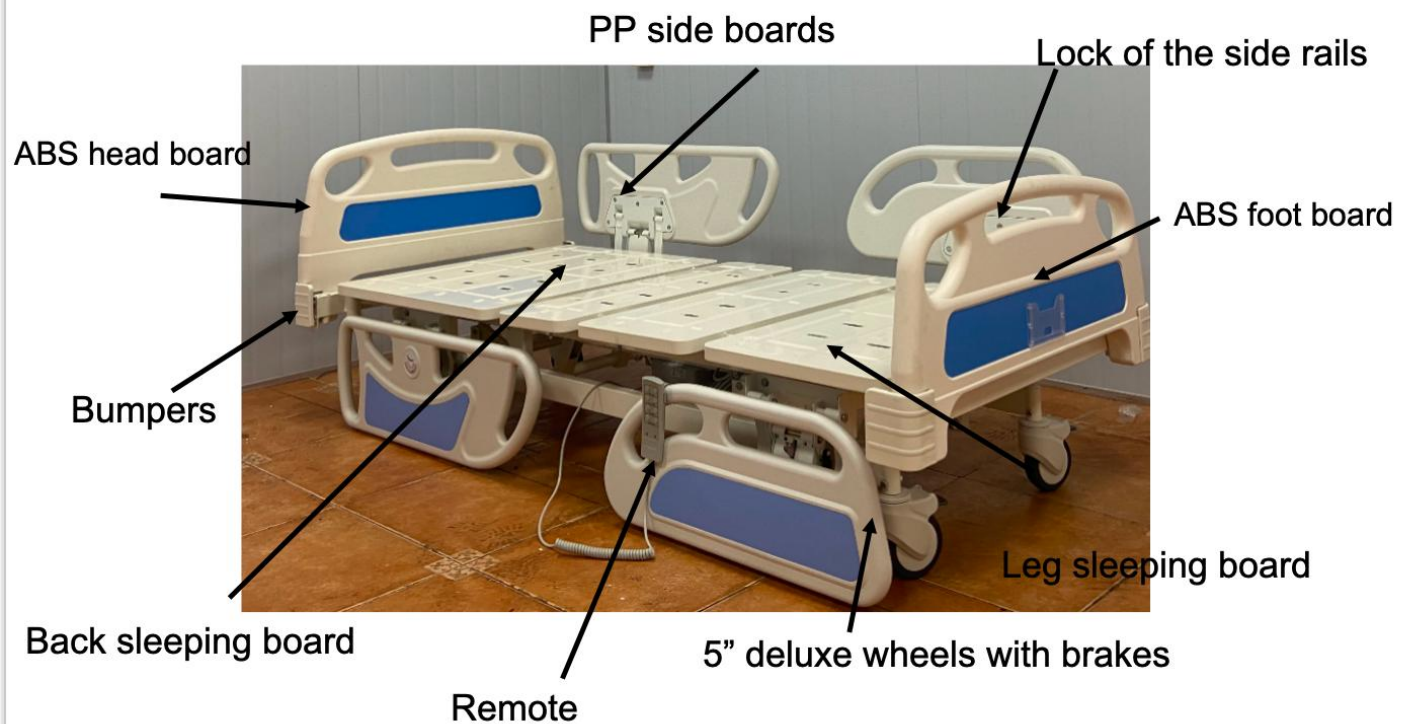

PARTS DESCRIPTION

The model shown above features the material of ABS head and foot board and the bed frame of steel powder coating, side rails are ABS. All motors are fixed underneath of the bed. And the motor is Denmark brand LINAK.

OPERATION OF THE HEAD AND FOOT BOARDS

The ABS Head and Foot Boards will be packed separately with the bed frame. Please fix the Head and Foot Boards as the following steps



Step No.1: Flick out the plastic switch



Step No.2: Focus on the two bars with the two steel mouths



Step No.3: Push down the board and it will be locked automatically.



WHEN DETATCH THE HEAD AND FOOT BOARD,FLICK OUT THE PLASIC SWITCH AND PULL UP THE BOARD.

WHEN YOU NEED TO LOCK UP WITH THE BOARDS,FLICK IN THE PLASTIC SWITCH.

OPERATION OF THE BED RAILS

The Side rails will be packed with the bed frame.Please fix the Side rail as the following steps:



button




Step No.1:When pulling up the rails,pull tight with the button and loose it when move to the top;When put down the rails, pull tight with the button and pull the rails down direction.

USING THE MOTORCONTROL

Use the **motor control** to activate the functions of the Backrest (from 0°to 75°);Footrest (from 0°to 45°)and the Bed Elevating (from 450mm to 740mm).trendelenburg and reverse trendelenburg 12°.

Please see the below instructions before using the control:

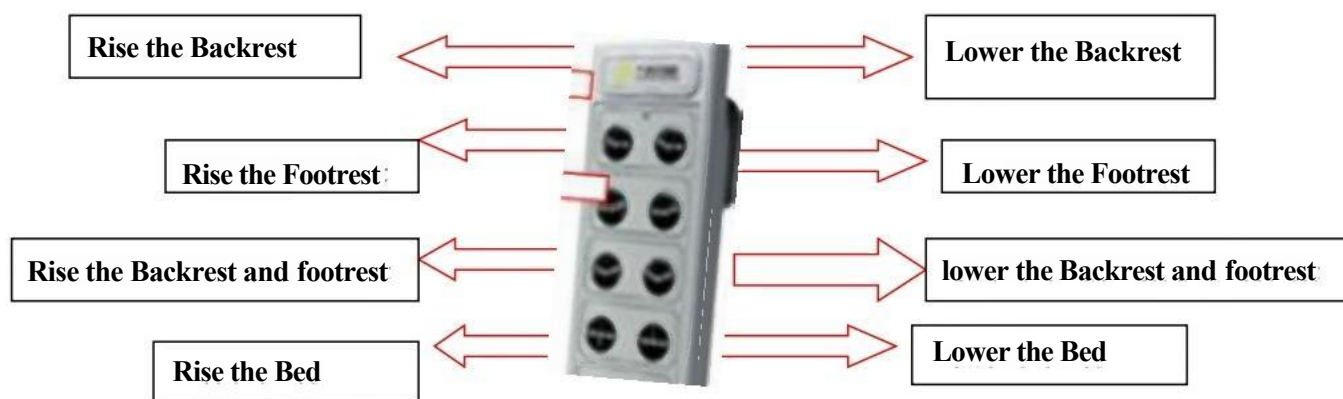

Electric Specification:

Voltage: 220V±10%,50-60Hz. AMPS: 1.6AMPS, 3.0AMPS at the most

Protection Class: IP54 Output Voltage: DC24V

Duty Cycle: Max. 10% or 6mins./1 hour in continuous use.

Ambient Temperature: +5°C to 40°C

The Electric bed with back-up batter, when there is no Electric power ,you can use the battery for 30 times including up-down of head, foot ,HI-LO. And the battery need to be charging for 5 hours.

CARE AND MAINTENANCE

Clean every part of the electric hospital bed frequently.

If the user makes biologic pollution, please use 75% alcohol to sterilize and clean it.

Manufacture: Topmost Medical Technology Co.,Ltd

Address: Address: No. 94, Middle of GaoMing Avenue, YangHe , GaoMing , Foshan city, China.

Telephone Number: +86-75726600229

- Before each use of the bed, the breaks of the castors and the fixed positions of the rails and head and foot boards should be checked.
- The bed should be stored in a dry environment, away from direct sunlight, when in storage the bed rails and the head and foot boards can be detached.
- The bed should be kept clean and dust free. This can be done with a duster or damp cloth.
- The user should routinely check the following items. It is recommended that a local dealer services the bed annually, where these items should be repaired, replace, adjusted and/or lubricated if required:
 1. Castors.
 2. Motors.
 3. Plastic switch of the Head and Foot Boards.
 4. Pp guardrail's Button
 5. Screws.

SPECIFICATOIN

Model	Dimensions
ABS Head Board (LxWxH)	910x530x66mm
ABS Foot Board (LxWxH)	910x500x66mm
Aluminum Side Rail (LxWxH)	L570*W300mm
The Mattress (LxWxH)	1950x840x80mm
The Bed (LxWxH)	2120x1030x450—740mm
Weight of the Bed	120kg
Capacity of the Bed	250kg

Notice:

Do not exceed the weight capacity stated above. Failure to do so could result in damage or injury.

WARRANTY

There is a comprehensive twelve-month warranty from the date on which your new bed is delivered. The warranty covers the bed for repairs or replacement during this period. For more detail, please see the warranty conditions overleaf:

- Any work or replacement part installation must be carried out by an authorized dealer/service agent.
- To apply the warrant should your bed require attention please contact the outlet from which you purchased the bed.
- Should any part of the bed require repair or full or part replacement, as a result of manufacturing or material defect within the warranty period, parts will be supplied free of charge. Note: The guarantee is not transferable.
- Any repaired or replaced parts will be covered by the balance of the warranty period on the bed.
- Parts replaced after the original warranty have expired will be covered by a three-month warranty.
- Consumable items supplied will not generally be covered during the normal warranty period unless such items require or replacement clearly as a direct result of a manufacturing or material defect. Such items include(among others) : castors.
- The above warranty conditions apply to brand new beds. Second-hand beds supplied directly by the authorized dealer carry a Six-month warranty period. If you are unsure whether your bed is covered contact your dealer.
- Under normal circumstances, no responsibility will be accepted where the bed have required assistance as a direct result of:
 - The bed part not having been maintained in accordance with the manufacturers recommendations
 - Failure to use the manufacturer specified parts
 - The bed or part having been damaged due to neglect, accident or improper use
 - The bed or part having been altered from the manufacturer specification or repairs having been attempted before the dealer is notified.

In the event of your bed requiring attention, please contact your service agent/dealer and give all relevant details so they can act quickly.